

3M™ V.A.C.® Ulta 4 Therapy System

Advanced technology made simple

3M™ Veraflo™ Therapy

Negative Pressure Wound Therapy with automated instillation and removal of topical wound cleansers.

- 1 Confirm Settings screen**
Select OK to accept default settings.
- 2 Fill Assist screen**
Fill Assist procedure will begin (could take up to 2 1/2 minutes to prepare the dressing seal to minimize solution leak potential).
- 3 Fill Assist screen**
Select Start/Stop to begin instilling fluid into the dressing.
- 4 Fill Assist screen**
Select Start/Stop again to stop instilling fluid into dressing. Select OK to accept settings and begin Veraflo Therapy.
- 5 Home screen - Veraflo Therapy**

3M™ V.A.C.® Therapy

Negative Pressure Wound Therapy designed to deliver the wound healing benefits of V.A.C.® Therapy with patented 3M™ SensaT.R.A.C.™ Technology.

- 1 Confirm Settings screen**
Select OK to accept default settings.
- 2 Seal Check™ Leak Detector screen**
Drawdown begins.
- 3 Home screen - V.A.C.® Therapy**

3M™ Prevena™ Therapy

Negative Pressure Therapy designed specifically for the management of closed surgical incisions.

- 1 Confirm Settings screen**
Select OK to accept default settings.
- 2 Seal Check™ Leak Detector screen**
Drawdown begins.
- 3 Home screen - Prevena Therapy**

3M™ AbThera™ Open Abdomen Negative Pressure Therapy

Negative Pressure Therapy designed to facilitate management of the open abdomen and help achieve primary fascial closure.

- 1 Confirm Settings screen**
Select OK to accept default settings.
- 2 Seal Check™ Leak Detector screen**
Drawdown begins.
- 3 Home screen - AbThera Therapy**



To learn more about the benefits of 3M™ V.A.C.® Ulta 4 Therapy System, please contact your local representative.

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

Important: This diagram is provided as a basic overview only. Refer to the 3M™ V.A.C.® Ulta Therapy System User Manual and the 3M™ V.A.C.® Ulta Therapy System Safety Information Sheet for detailed instructions, including important safety information. For medical emergencies, consult treating physician or call emergency services. Carefully read and follow all literature and labeling accompanying product.